LAICA™

2

MD6026

ANNI DI **GARANZIA**
ANS DE **GARANTIE**
YEARS **GUARANTEE**
ANOS DE **GARANTIA**
ANOS DE **GARANTIA**
JAHRE **GARANTIE**
ΧΡΟΝΙΑ **ΕΓΓΥΗΣΗ**
ANI DE **GARANȚIE**

Data - Date

| | | | | | | | | | |
|--|--|--|--|--|--|--|--|--|--|
| | | | | | | | | | |
|--|--|--|--|--|--|--|--|--|--|

Timbro rivenditore - Cachet du revendeur
Retailer's stamp - Sello del revendedor - Carimbo do revendedor
Stempel des Händlers - Σφραγίδα καταστήματος - Stampilă distribuitor

LAICA™

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MD6026

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Device for ultrasound aerosol therapy with first rate design and maximum technology characterized by an extreme silentness. The aerosol therapy is efficient for the treatment of pulmonary disorders, tracheitis, asthma, bronchitis, sinusitis and allergies; moreover, it can alleviate the disorders caused by colds and allergic irritations and mitigate the stimulus of cough, pain and sense of compression in the area of the paranasal sinuses easing the expectoration. It is excellent for domestic purposes, practical and simple to use.

It is manufactured in compliance with current EC standards concerning constructive criteria, for the safety of equipment for electromedical use (EN 60601-1 Standard). Through the ultrasound aerosol the preparations for inhalations are transformed by the oscillation of a piezoelectric crystal present within the device, in micro-drops of liquid dispersed in air.

These drops, having sizes included within 1 and 5 µm, succeed in penetrating deeply into the upper and lower respiratory tract, bringing therefore the active ingredients to the right places and in a targeted way.

Its reduced dimensions make this product extremely accessible for those who require to use it several times a day and for those who travel often.

Doctor's instructions should always be followed for the type of pharmaceutical to be used, the dose and frequency of the inhalation treatments.

Laica S.p.A. - Viale del Lavoro, 10 - Fraz. Ponte 36021 Barbarano Vicentino (VI) - Italy

 0434 (Dir. 93/42/EEC)

Before use it is important to read the instructions and warnings contained in this booklet carefully and keep it in a safe place for future reference.

GENERAL WARNINGS

Using any electrical appliance entails compliance with a number of fundamental rules:

- Unplug the appliance from the mains straight after use and before commencing cleaning or inserting/removing accessories.
- Do not unplug by tugging on either power cord or appliance.
- If an electrical appliance falls into the water, do not attempt to retrieve it; instead, unplug immediately.
- Do not use the appliance while bathing or showering.
- Never immerse the appliance in water or other liquids.
- **Never leave the appliance working unattended, and never allow it to be used by children, handicapped or disabled individuals.**
- **Babies and people with impaired physical, sensory or mental skills or inexperienced people shall use the product only if prior**

trained on the safe use of the same and only if informed on the hazards linked to the product itself.

- This appliance must be used solely for the purpose for which it was designed and in accordance with the operating instructions. Any other use shall be considered improper and hence hazardous.

The manufacturer cannot be held responsible for any damage resulting from improper or incorrect use.

- Before using the appliance, make sure it is intact and does not feature any visible signs of damage. If in any doubt, do not use the appliance and contact your dealer.
- In the event of a fault and/or irregular operation, turn the appliance off at once without tampering with it. Always contact your dealer for repairs.
- Keep the appliance away from sources of heat.
- Make sure your hands are dry when unplugging or plugging in the appliance and when turning the function control knob.
- This device shall be used only with the voltage stated in this booklet.
- Device with enclosure not protected against the penetration of liquids.
- Do not use the device close to high-frequency electromagnetic emitters.
- Keep the aeration opening free from dust and impurities. Avoid leaning the device on soft surfaces (i.e. bed or pillows).
- When not using the device, store it where it is protected from dust and impurities and out of direct sunlight and keep away from children and pets.
- Keep all parts of the packaging as well as the air hose and adaptor away from children (risk of suffocation and strangling)

IMPORTANT WARNINGS

- In case of diabetes or other illnesses, it is suggested to consult your physician before undergoing the inhalation therapy.
- Pregnant women should consult their physician before undergoing the inhalation therapy.
- Before using a preparation, clear with your physician, if it is suitable for the person to use it. Use only medicines prescribed or suggested by your physician.

IMPORTANT NOTES FOR BEST OPERATION

- 1) Do not use the equipment if the nebuliser chamber is without water or if the water level is below the indicated level.
- 2) The disposal medical cup is **SINGLE USE**; replace it after each use to guarantee best nebulisation and to avoid damage to the equipment.

- 3) Do not use oily medicines with a viscosity over 3 cPs (centipoise), the unit to measure viscosity, for example cortisone, as it could damage the transducer.
- 4) Do not use essential oils (such as eucalyptus essence), active ingredients with solid elements (suspended) and viscous solutions as they could damage the transducer.
- 5) **After every use**, the nebuliser chamber (no.8 exploded view) and the transducer (no.9 exploded view) should be cleaned with a cloth dipped in pure alcohol.
- 6) Wait at least 15 minutes from one use to the next.
- 7) Clean the top cover (no.5 exploded view) of the device handling the flow duct with care (no.6 exploded view).

ACCESSORIES / COMPONENTS

- 1) Main unit
- 2) "ON/OFF" key
- 3) Operation warning light ("ON/OFF")
- 4) Water tank
- 5) Upper cover
- 6) Stream conveyor
- 7) Disposable medicine cup (8cc) with spare parts
- 8) Nebulization chamber
- 9) Transducer
- 10) Air connection pipe
- 11) Masks connector
- 12) Mask for adults
- 13) Mask for children
- 14) Mouthpiece
- 15) Adaptor plug
- 16) Adaptor

INSTRUCTIONS FOR USE

- 1) Before each use, carefully clean and disinfect the water tank, the cover, the masks connectors, the air connection tube, the mouthpiece and the masks according to paragraph "Cleaning and maintenance".
- 2) Place the device on a flat surface.
- 3) Remove the cover. The tank holds 15 cc. Fill with tap water up to the red level indicator. This water is used as liquid conducting to the drug the ultrasonic waves and is never nebulized. Do not put the tank directly under the tap to fill it with water. It is strictly forbidden to use salt solution and water with any chemical element.
Do not pour drugs directly in the tank.
- 4) Fill the nebulizer with the drug, then place it in the water tank according to figure 3. Pour the drug paying attention not to exceed the maximum capacity (8cc). Follow the instructions for use and the dosing of the medicinal product you are using. If such product has to be diluted with an isotonic solution of sodium chloride, pour first of all the

solution in the nebulizer and then add the product according to the stated dosing.

- 5) Close the tank with the proper cover.
- 6) Now, insert the pipe and the mouthpiece or the connector and the mask for adults/children in the cover, according to figure 6.

In case of inhalation through mouthpiece:

In case of troubles at low respiratory system, for example cough or bronchial infections, use the mouthpiece.

- Sit upright.
- Connect the air connection tube in the cover and in the other end insert the mouthpiece.
- Tighten the mouthpiece well with the lips.
- Inhale slowly and deeply through the mouth and exhale through the nose. To increase the efficacy of the therapy, hold your breath for a while after having inhaled. During the treatment, remain quiet and relaxed. Breathe regularly and not too quickly. If you wish to interrupt the inhalation shortly, remove the mouthpiece from the mouth and then reinsert it.

In case of inhalation with mask:

The mask for inhalations is suitable for the treatment of the nasal and pharyngeal area.

- Sit upright.
- Connect the air connection tube in the cover and in the other end insert the fitting and the mask.
- Keep the mask lightly pressed against the nose; the mask has to cover mouth and nose without squashing them.
- Inhale and exhale slowly and deeply through the nose. To increase the efficacy of the therapy, hold your breath for a while after having inhaled. During the treatment, remain quiet and relaxed. Breathe regularly and not too quickly. If you wish to interrupt the inhalation shortly, remove the mask for a while and then place it again on nose and mouth.

- 7) Connect the transformer to the device and then to the power supply plug paying attention that the voltage is right.
- 8) Switch on the device pressing the "ON/OFF" key. Make sure that the warning light switches on (blue colour). During inhalation, the device must compulsorily remain in vertical position: this assures its correct operation, as well as the maximum efficacy of therapy. Immediately interrupt the treatment if you feel unease during the aerosol-therapy.
- 9) The device has an automatic stop function, which deactivates the device when the medicine is almost exhausted or after 13 minutes of inhalation. The device can be switched off at any time by pressing

the "ON/OFF" switch.

IMPORTANT:

As with most equipment for aerosol therapy, at the end of the inhalation treatment, a certain amount of the pharmaceutical will remain in the nebulization ampoule: this is perfectly normal. This amount of pharmaceutical, also known as residual volume, cannot be nebulized.

- 10) Let the device cool down for some minutes, turn the device upside down or use an absorbing cloth to remove the residual liquid and perform the cleaning procedures according to paragraph "Cleaning and maintenance".
- 11) Wait at least 15 minutes from one use to the next.

REPLACEMENT OF THE PHARMACEUTICAL AMPOULE

The disposal medicine cup is **SINGLE USE**; replace it after each use to guarantee best nebulisation and to avoid damage to the equipment.

Within the pack you will find ampoules which you can use for replacement purposes.

Other ampoules can be purchased from your retailer.

MICROBIC CONTAMINATION

In the presence of conditions carrying the risk of microbial infection and contamination, personal use of the accessories only is advised (always consult a doctor).

CLEANING

1) Caution! Hot surface!

Before cleaning the device, let it cool since the transducer heats during the use.

- 2) After having disconnected the socket, remove the cover and empty the tank from any eventual residuals.
- 3) **After every use**, the nebuliser chamber (no.8) and the transducer (no.9) should be cleaned with a cloth dipped in pure alcohol. Clean the top cover (no.5) of the device with a damp or vinegar-dipped cloth to remove all residues. During cleaning, handle the flow duct with care. (no.6)
- 4) Clean the external part of the device using a soft cloth soaked with water and neutral soap (**do not clean the main unit by placing it under running water and pay attention that no liquid penetrates into the device**).
- 5) In case of daily use, it is suggested to disinfect or sterilize the accessories all the days to limit the risk of infections.
In this case, place all accessories, except for the air connection tube, in boiling water for 10 minutes and then let them dry completely before reusing them. Wash air connection tube carefully with drinking water.

Leave to dry naturally.

- 6) Disinfect or sterilize all components, except for the air connection tube, in boiling water for 10 minutes also before using the device for the first time. Wash air connection tube carefully with drinking water. Leave to dry naturally.
- 7) To disinfect the device, it is also possible to use 70% alcohol (or a disinfecting spray with alcoholic base). Check that within the device there are not residual parts of the disinfectant.
- 8) The device, the tank and all accessories must be completely dry before using them.
- 9) Store the device in a fresh and dry place.

TECHNICAL SPECIFICATIONS

- Power supply:
INPUT: 100-240V ~ 50-60Hz, 0.15A;
OUTPUT: 13,5V, 1A
- **Operating conditions:**
Temperature: min. 5°C; max. 40°C
Relative humidity: 15-93%, in absence of condensate
- **Storing conditions:**
Temperature: -25°C ~ +70°C
Relative humidity: less than 93%, in absence of condensate
- Ultrasound frequency: 1.7 MHz
- Nebulization capacity: about 0.4 ml/minute
- Percentage of aerosol of the inhalation: 60% of the particles < 5 µm
- Medium size of the particles: 3.4 µm
- Maximum volume of the inhaling product:
max. 8 ml = 8 cc
- Size: 235 x 145 x 285 mm
- Weight: 350 g
- ☺ Device of BF type
- ☑ Device of second class
- ☠ Caution: check the instructions for use

IP21: Degree of protection of electrical equipment covers, where the first figure indicates the degree of protection against penetration by solid foreign bodies (from 0 to 6) and the second figure the degree of protection against penetration by liquids (from 0 to 8).

ELECTROMAGNETIC COMPATIBILITY

This device complies with current regulations relating to electromagnetic compatibility (EMC) and has been designed to be used in the home.
Its emissions are extremely small and are unlikely to generate interference with other equipment.
In the event that it is to be used in close proximity to other equipment, it is recommended to follow

the instructions given in the tables at the end of the instructions for use.

Certain mobile communications devices and radio systems may have an effect on the operation of this device.

In cases where there are anomalies during operation, it is recommended to distance the device from any other equipment that could cause electromagnetic interference and check for the restoration of performance.

In all cases, in the event of doubt, contact Customer Support or see the tables relating to electromagnetic emissions at the end of these instructions.

DISPOSAL PROCEDURE (Dir. 2012/19/Eu-WEEE)

 The symbol on the bottom of the device indicates the separated collection of electric and electronic equipment.

At the end of life of the device, do not dispose it as mixed solid municipal waste, but dispose it referring to a specific collection centre located in your area or returning it to the distributor, when buying a new device of the same type to be used with the same functions.

If the appliance to be disposed of is less than 25 cm, it can be returned to a retail location that is over 400 m² without having to purchase a new, similar device.

This procedure of separated collection of electric and electronic devices is carried out forecasting a European environmental policy aiming at safeguarding, protecting and improving environment quality, as well as avoiding potential effects on human health due to the presence of hazardous substances in such equipment or to an improper use of the same or of parts of the same.

Caution! The wrong disposal of electric and electronic equipment may involve sanctions.

WARRANTY

This equipment is covered by a 2-year warranty starting from the purchase date, which has to be proven by means **of the stamp or signature of the dealer and by the fiscal payment receipt that shall be kept enclosed to this document**. This warranty period complies with the laws in force and applies only in case the consumer is an individual. Laica products are designed for domestic use only; therefore, their use for commercial purposes is forbidden.

The warranty covers only production defects and does not apply, if the damage is caused by accidental events, misuse, negligence or improper use of the product. Use the supplied accessories only; the use of other accessories invalidate the warranty cover.

Do not open the equipment for any reason whatsoever;

the opening or the tampering of the equipment, definitely invalidate the warranty. Warranty does not apply to parts subject to tear and wear along the use, as well as to batteries, when they are included.

Warranty decays after 2 years from the purchase date. In this case, technical service interventions will be carried out against payment. Information on technical service interventions, under warranty terms or against payment, may be requested by writing at info@laica.com. Repair or replacement of products covered by the warranty terms are free of charge. In case of faults, refer to the dealer; DO NOT send anything directly to LAICA.

All interventions covered by warranty (including product replacement or the replacement of any of its parts) shall not be meant to extend the duration of the original warranty period of the replaced product.

The manufacturer cannot be made liable for any damages that may be caused, directly or indirectly, to people, things and pets resulting from the inobservance of the provisions contained in the suitable user manual, especially with reference to the instructions related to installation, use and maintenance of the equipment.

Being constantly involved in improving its own products, Laica reserves the right to modify its product, totally or partially, with reference to any production need, without that this involves any responsibility whatsoever by Laica or its dealers.

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- MD6026 needs special precautions regarding EMC and needs to be installed and put into service according to the EMC information provided in the ACCOMPANYING DOCUMENTS;
- Portable and mobile RF communications equipment can affect MD6026.
- Warning that the use of accessories, transducers and cables other than those specified with the exception of transducers and cables sold by the manufacturer of the MD6026 as replacement parts for internal components, may result in increased EMISSIONS or decreased IMMUNITY of the MD6026.
- Warning that the MD6026 should not be used adjacent to or stacked with other equipment.
During Voltage Dips test, the product will change its operation state from on to off, after test, it needs operator to press on/off button to recover.
This phenomenon does not affect BASIC SAFETY or ESSENTIAL PERFORMANCE, according the standard, it's deemed to comply with the requirement.

| Guidance and manufacturer's declaration - electromagnetic emissions | | |
|--|-------------------|---|
| The MD6026 is intended for use in the electromagnetic environment specified below. The customer or the user of the MD6026 should assure that it is used in such an environment. | | |
| Emissions test | Compliance | Electromagnetic environment – guidance |
| RF emissions CISPR 11 | Group 1 | The MD6026 uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment. |
| RF emissions CISPR 11 | Class B | The MD6026 is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes. |
| Harmonic emissions IEC 61000-3-2 | Class C | |
| Voltage fluctuations/ flicker emissions IEC 61000-3-3 | Complies | |

| Guidance and manufacturer's declaration - electromagnetic immunity | | | |
|--|--|--|---|
| The MD6026 is intended for use in the electromagnetic environment specified below. The customer or the user of the MD6026 should assure that it is used in such an environment. | | | |
| Immunity test | IEC 60601 test level | Compliance level | Electromagnetic environment – guidance |
| Electrostatic discharge (ESD) IEC 61000-4-2 | ±6 kV contact ±8 kV air | ±6 kV contact ±8 kV air | Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30 %. |
| Electrical fast transient/burst IEC 61000-4-4 | ±2 kV for power supply lines | ±2 kV for power supply lines | Mains power quality should be that of a typical commercial or hospital environment. |
| Surge IEC 61000-4-5 | ±1 kV line(s) and neutral | ±1 kV line(s) and neutral | Mains power quality should be that of a typical commercial or hospital environment. |
| Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11 | <5 % UT (>95 % dip in UT) for 0,5 cycle 40 % UT (60 % dip in UT) for 5 cycles 70 % UT (30 % dip in UT) for 25 cycles <5 % UT (>95 % dip in UT) for 5s | <5 % UT (>95 % dip in UT) for 0,5 cycle 40 % UT (60 % dip in UT) for 5 cycles 70 % UT (30 % dip in UT) for 25 cycles | Mains power quality should be that of a typical commercial or hospital environment. If a dips or an interruption of mains power occurs, the current of the MD6026 may be dropped off from normal level, it may be necessary to use uninterruptible power supply or a battery. |
| Power frequency (50/60 Hz) magnetic field IEC 61000-4-8 | 3 A/m | Not applicable | Not applicable |
| NOTE UT is the a.c. mains voltage prior to application of the test level | | | |

Guidance and manufacturer's declaration - electromagnetic immunity

The MD6026 is intended for use in the electromagnetic environment specified below.
The customer or the user of the MD6026 should assure that it is used in such an environment.

| Immunity test | IEC 60601 test level | Compliance level | Electromagnetic environment – guidance |
|-------------------------------|-----------------------------|------------------|--|
| Conducted RF IEC 61000-4-6 | 3 Vrms 150 kHz to 80 MHz | 3 Vrms | <p>Portable and mobile RF communications equipment should be used no closer to any part of the MODEL MD6026, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.</p> <p>Recommended separation distance $d = 1,2\sqrt{P}$ $d = 1,2\sqrt{P}$ 80 MHz to 800 MHz $d = 2,3\sqrt{P}$ 800 MHz to 2,5 GHz</p> <p>where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation Distance in metres (m). Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey, a should be less than the compliance level in each frequency range. b Interference may occur in the vicinity of equipment marked with the following symbol:</p>  |
| Radiated RF IEC 61000-4-3 | 3 V/m 80 MHz to 2.5 GHz | 3 V/m | |

Recommended separation distances between portable and mobile RF communications equipment and the MD6026

The MD6026 is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the MD6026 can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the MD6026 as recommended below, according to the maximum output power of the communications equipment.

| Rated maximum output power of transmitter W | Separation distance according to frequency of transmitter m | | |
|---|---|--|---|
| | 150 kHz to 80 MHz $d = 1,2\sqrt{P}$ | 80 MHz to 800 MHz $d = 1,2\sqrt{P}$ | 800 MHz to 2,5 GHz $d = 2,3\sqrt{P}$ |
| 0.01 | 0.12 | 0.12 | 0.23 |
| 0.1 | 0.38 | 0.38 | 0.73 |
| 1 | 1.2 | 1.2 | 2.3 |
| 10 | 3.8 | 3.8 | 7.3 |
| 100 | 12 | 12 | 23 |

For transmitters rated at a maximum output power not listed above, the recommended separation distance d in metres (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.
 NOTE 1 At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.
 NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

NOTE 1 At 80 MHz and 800 MHz, the higher frequency range applies.
 NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

- a. Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the MODEL MD6026 is used exceeds the applicable RF compliance level above, the MODEL MD6026 should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating the MODEL MD6026.
- b. Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3 V/m.